## english


# Pre-Seed®

## Vaginal lubricant when trying to conceive

- Gives an optimal sperm environment
- Doesn't harm sperm or embryos
- Clinically tested and Doctor recommended
- pH balanced to Match Fertile Cervical Mucus

### Directions for Using Pre-Seed® with Disposable Applicators:

1. Remove seal from tube before initial use.
2. Pre-Seed® can be applied up to 15 minutes prior to intercourse. Applying it before you begin making love allows the moisture to disperse throughout the vagina and allows more spontaneity for you as a couple.
3. Attach the applicator to the Pre-Seed® tube by twisting the end of the applicator firmly onto tube (see picture).

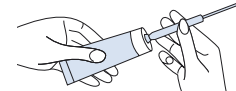4. Gently squeeze the Pre-Seed® into the applicator, and continue squeezing until the applicator is full or the desired amount is present in the applicator. Most women will choose the 3 gm fill line to meet their product needs. Separate the applicator from the tube. Be sure and keep the open end of the applicator upright so product is not lost. After each use, replace the cap and roll up the tube from the bottom. Do not leave Pre-Seed® uncapped between uses.

5. Do not store Pre-Seed® in the applicator for more than 30 minutes prior to use. This means

if you draw Pre-Seed® into the applicator but do not insert it intravaginally within 30 minutes, the unused Pre-Seed® needs to be discarded and the applicator disposed of.

6. To apply Pre-Seed®, gently insert the applicator deep into the vagina. This can be done while standing, lying down or sitting (such as on the toilet).
7. Holding the barrel of the applicator, slowly push the plunger all the way in to release the lubricant into the vagina (see picture).

8. Remove both parts of the applicator from the vagina.
9. In order to get the right amount of moisture for your body\*, bear down slightly after depositing the product. This will expel any excess Pre-Seed® that your body doesn't need. If desired, use a tissue to lightly wipe off any extra product that is on the vulva.
10. Discard applicator following use. DO NOT REUSE APPLICATORS.

\* If this application method doesn't provide enough moisture, we suggest you apply Pre-Seed inside the vagina (as shown) while lying down in bed, IMMEDIATELY before intercourse. You may also want to squeeze a small amount of Pre-Seed from the tube onto your fingertips, and apply directly to external genitalia. If this application method provides too much moisture for you, apply the product earlier before intercourse (e.g., up to an hour), or apply fewer grams of product. Also, be sure to bear down to expel extra Pre-Seed after application, and wipe off excess product from the vulva area

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the product. If symptoms persist contact your physician.

- Keep out of reach of children.
- **Do not use product if active infection or vaginal trauma is present.** If you have had recent surgery please seek medical advice before using.

**Directions for Using Pre-Seed® Externally:** Squeeze a small amount of Pre-Seed® (size of a quarter or teaspoon) onto your fingertips and apply to genital area. Vary amount to achieve desired lubrication. Pre-Seed's mild ingredients require you to apply a bit more Pre-Seed initially than you might with other products, in order to saturate the skin and provide longer lasting lubrication.

**Ingredients:** Purified Water, Hydroxyethylcellulose, Pluronic, Sodium Chloride, Sodium Phosphate, Carbomer, Methylparaben, Sodium Hydroxide, Arabinogalactan, Potassium Phosphate, Propylparaben

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